# DELABIE

# BIOFIL

NT EMAR 423\_1

Indice 1

10/2024

**FR**

## BIOFIL - FILTRES TERMINAUX ANTI-TOUS GERMES

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The medical devices covered by this installation guide are all-germ filters for taps and/or showers. They are commonly referred to as "**BIOFIL tap filters**" for microfiltration cartridges and "**BIOFIL screw-on shower filters**" for shower heads fitted with a microfiltration cartridge.

They are installed at the end of the drinking system (on taps and showers) and serve as **physical barriers in the event of a bacterial alert** (*Legionella pneumophila*, *Pseudomonas aeruginosa*, and other Gram-negative bacilli, non-tuberculous Mycobacteria, fungi, protozoa and other particles from the water distribution system – absolute filtration grade of 0.2 µm) for an indicative duration of use of 2, 3 or 4 months. The protection of immunocompromised patients and vulnerable people is ensured thanks to the microfiltration cartridge. These medical devices are for **single use only**.

They are intended to be installed in hospitals or medical care facilities (retirement homes, specialised centres), as well as in public buildings such as leisure or sports centres, hotels, etc.

## 2 REFERENCES

- BIOFIL screw-on shower filters: 20261, 20361, 20461, 30261, 30361, 30461.
- BIOFIL tap filter with laminar flow: 20251, 20351, 20451, 30251, 30351, 30451.
- BIOFIL tap filter with shower spray: 20250, 20350, 20450, 30250, 30350, 30450.

**To guarantee optimal performance of the BIOFIL filters, please comply with the installation protocol (see 3) and the precautions for use (see 7).**

The list is available on the website: [www.delabie.com](http://www.delabie.com) or from our customer service teams:

**For the UK market only:** Tel. 01491 821821 - email: [technical@delabie.co.uk](mailto:technical@delabie.co.uk)

**For all other markets:** Tel. +33 (0)3 22 60 22 74 - email: [sav@delabie.co.uk](mailto:sav@delabie.co.uk)

## 3 INSTALLATION

### Warnings:

Wash and disinfect your hands before installing the filter to limit the risk of contamination or retro-contamination.

Check that the individual packaging sachet is not punctured or torn and that the filter is not damaged.

If necessary, dispose of the filter as its safety and performance are no longer guaranteed.

Check that the expiry date has not passed. If necessary, discard the filter because its safety and performance are no longer guaranteed.

### Preliminary preparation of the taps:

- The screw-on shower filter: quickly screw clockwise all the way onto any shower hose complying with standard NF EN 1113. The length of the hose must be adapted to avoid any contact between the shower filter and the ground. The o-ring provided must be installed to ensure it is fully sealed and prevent possible leaks. Each time the shower head is changed, the o-ring should be replaced.
- The tap filter: it must be fitted using the push-fit connectors recommended by DELABIE. The push-fit connectors allow the filter to be connected to and disconnected from the tap easily.  
Visit the website or contact us for a list of available connectors.

### Installation:

1. Turn on the tap to flush it before installation to avoid early clogging.
2. Isolate or shut-off the water supply to electronic taps.
3. For first-time installations:
  - Tap filters: use an adapter.
  - Screw-on shower filters: remove the original screw-on shower head.
4. If replacing a worn filter: remove the filter.
5. **Wash and disinfect your hands.**
6. Partially open the individual sachet.
7. While keeping the filter in the bag to prevent contamination of the water outlet, screw the shower head filter to the end of the hose or clip the cartridge filter on to the adapter.
8. Remove the individual sachet.
9. Turn the water on for a few seconds to check that the filter and fittings remain perfectly sealed.
10. In case of leaks, repeat the procedure. Do not re-use a filter that has been connected.
11. Note the commissioning and the theoretical decommissioning date (commissioning date + theoretical duration of use) on the filter and on the traceability label.

### Removal:

1. Shut off the water supply.
2. Release the used shower head filter by unscrewing it from the end of the hose or unclip the shower head filter or tap filter from the adapter.
3. Dispose of the filter as ordinary household waste. Never re-use a filter that has been connected. As the filter cannot be dismantled, the manufacturer cannot guarantee that the fibres remain in good condition.  
Installation and use in a dirty or uncontrolled environment can be a source of retro-contamination. Discard and replace any filter that appears visually soiled or contaminated.

## 4 CLEANING

The outer surfaces of the filter (except for the water outlet) can be cleaned with a single-use disinfectant wipe, following the cleaning protocols commonly implemented in healthcare establishments (wear protective equipment, specific protocol, etc.).

## 5 TECHNICAL PERFORMANCE AND EXPOSURE TO CHEMICAL AND THERMAL TREATMENTS

### Conditions of use:

The performance of BIOFIL filters is guaranteed when the pressure and temperature of the water in the system are respectively between 1 and 5 bar/5°C and 42°C. The the water flow rates measured at the outlet are as follows:

	BIOFIL filter				
	Water pressure (bar)				
	1	2	3	4	5
Flow rate (L/min) - 2M tap	3	4.4	5.5	6.5	7.4
Flow rate (L/min) - 3M tap	4.6	6.6	7.6	8.6	9.4
Flow rate (L/min) - 4M tap	4.6	6.5	7.6	8.5	9.4
Flow rate (L/min) - 2M shower head	6	8.8	11.2	12.8	13
Flow rate (L/min) - 3M shower head	6.2	9.2	11.2	12.4	13.2
Flow rate (L/min) - 4M shower head	6.5	9.6	11.8	13.2	14.1

### Exposure to chemical and thermal treatments:

The filters can tolerate the following treatments:

- Thermal treatment at 70°C for 30 minutes (maximum cumulative).
- Chemical treatment: active chlorine.
  - 2M filter: 100 ppm for 2 hours.
  - 3M filter: 100 ppm for 3 hours.
  - 4M filter: 100 ppm for 4 hours.

Replace the filter if the chemical or thermal treatment exceeds these thresholds.

### Clogging tolerance:

If the water in the drinking water system has an unusual level of particles or microorganisms in suspension, it is possible to observe a decrease in the output flow rate before the theoretical decommissioning date (date of entry into service + theoretical duration of use).

When the flow rate becomes insufficient, replace the filter.

**Warning:** Clogging may occur before the theoretical decommissioning date, but it does not directly affect the patient.

**Advice:** In the case of early clogging, pre-filtration at different levels of the system is recommended to stop suspended solids and/or colloids.

## 6 DISPOSAL

Used filters do not pose a risk of infection. Like all bimaterial packages, they are classified as household waste. The instructions and the cardboard box are recyclable (paper, cardboard).

## 7 PRECAUTIONS FOR USE

- Wash and disinfect your hands before touching or cleaning a filter.
- Flush the tap before installing the filter to avoid early clogging.
- Do not use the filter if the packaging and/or the filter are damaged.
- Avoid collisions with or knocks to the filter. Replace if damaged.
- Discard and replace any filter that appears visually soiled or contaminated.
- Do not disconnect the filter while in use.
- The water outlet must not come into contact with skin or hair. In the case of contact, there is a risk of retro-contamination. We recommend replacing the filter.
- If there is a decrease in flow rate, we recommend replacing the filter.
- Once disconnected, the filter should be discarded (single use).
- The microfiltration cartridge must not be adapted, modified or opened.
- The filter must not be used for the production of water for pharmaceutical use.
- The filter does not change the potability of the water, and it does not modify its concentration of salts, metals, scale, chemicals, pesticides or dyes.
- Shower filters must always be repositioned on the support after use.

## 8 UNDESIRABLE SIDE EFFECTS

The studies carried out have not demonstrated any adverse effects on the health of patients.

## 9 PRODUCT COMPLIANCE

BIOFIL sterile filters:

- are sterilised by gamma irradiation in accordance with NF EN ISO 11137 standard. Do not re-sterilise.
- are class Is medical devices compliant with European Regulation EU 2017/745. CE marked 0459.
- have international drinking water compliance validations (ACS - France, KTW - Germany, WRAS - United Kingdom).

BIOFIL non-sterile filters:

- are class I medical devices which comply with European Regulation EU 2017/745. CE marked.
- have international drinking water compliance validations (ACS - France, KTW - Germany, WRAS - United Kingdom).

## 10 MANUFACTURER DETAILS

BIOFIL filters are manufactured by:

**AQUATOOLS** | 11, rue Lobineau | 75006 PARIS | France

Website: [www.aqua-tools.com](http://www.aqua-tools.com)

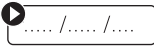
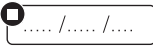
And are distributed by DELABIE:

1. For more information, visit our website: [www.delabie.com](http://www.delabie.com).  
or contact us: **tel.:** + 33 (0)3 22 60 22 74 - **E-mail:** [sav@delabie.fr](mailto:sav@delabie.fr).
2. The "Scientific and Technical Validation Guideline" relating to filters is available on request.
3. Paper versions of the installation guides are available on request.
4. Any serious incident must be notified to the manufacturer and to the competent authority of the Member State where the user and/or the patient resides.

### Reference:

A sterilising grade filter is defined in the "American Standard Test Method ASTM F838-20 and FDA - Guidance for Industry - Sterile Drug Products Produced by Aseptic Processing - Current Good Manufacturing Practice.

### Product label - meaning of the pictograms

		
Catalogue reference	Batch code	Expiration date
		
Device sterilised by irradiation	Do not re-sterilise	Do not use if the packaging is damaged.
		
Manufacturer's address	Meets general safety and performance requirements	Consult the precautions for use
		
Sterile barrier system only.	Medical Device	Unique Device Identifier
		
Do not reuse	Manufacturing date	Shower head filter
		
Tap filter	Shower spray	Laminar flow
		
Batch number	Commissioning date	Decommissioning date
		
Quantity	Global Trade Item Number	Use-by date after commissioning 2M: 2 months - 3M: 3 months - 4M: 4 months
		
Non-sterile device		

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